

MAS90 Magmobil

Frequency Reactions

The human and the animal organism react at various frequency bands which are relevant as follows:

Delta Band (1 – 3,5Hz):

Theta Band (4 – 7Hz):

Alpha Band (8 – 13 Hertz):

Beta Band (14 – 39Hz):

MAGmobil provides a total of 7 programs: in programs 1 to 4 is stored each of the 4 frequency bands.

Program 1 contains the Alpha Band

For applications involving: Disorders of the internal organs such as gall and urinary bladder, prostatism, nervousness, sleep disorders, venous illnesses, menstruation problems: please see also the preceding comments.

Program 2 contains the Beta Band

For applications involving: spasm release, increase in blood circulation levels, enhanced nutrient and oxygen uptake, stimulation of the metabolism system. The body can regenerate.

Program 3 contains the Theta Band

For applications involving: Inflammation and abrasion of the joints, tinnitus, lumbago, contusions, pain in the back/neck/shoulder, muscular tension.

Program 4 contains the Delta Band

For applications involving: Arthritis, Arthrosis, vertebral disk damage, fibromyalgia, scar and phantom pain, bone fractures and similar consequences of injuries.

Program 5 is tuned to the Schumann frequency (7,83Hz)

Investigations have demonstrated that the Schumann frequency is in exact resonance with the hippocampus frequency of humans and all mammals. This signal has been identified by NASA (Prof. Persinger), Prof. R. Wever and by the bio-physicist Dr. Wolfgang Ludwig independently of each other as "**biologically normal**" without which neither man nor beast can get by. The absence of this level of oscillation can cause disruption of the general well being factor such as pulse rate alterations and respiratory changes.

Program 6 is a magnetic field signal comprising several frequencies in which the frequency bands referred to above re-occur.

Program 6 works on the whole organism in order to increase the level of general well being.

Program 7 is a freely programmable program.

The application of the MAGmobil should take place, as far as time allows, once or twice daily. The duration of the application should be between 15 and 30 minutes.